Instructions for Use

Instructions for stainless steel instruments

This instrument is provided non sterile and should be cleaned and sterilized before the first use and each re-use.

Cleaning

To prevent stains, use mild detergents and distilled or demineralized water for cleaning.

Soak instrument in lukewarm water with mild detergent. Remove any organic matter while submerged with soft brush. Rinse thoroughly with distilled or demineralized water. Dry with compressed air or lint-free cloth.

Sterilization

Do not soak instrument in glutaraldehyde, chlorine or ammonium solution or sterilize by dry heat as this may cause damage to the instrument finish.

Prior to autoclaving, ensure that the instrument is free from any debris. Always autoclave instruments in an open position. Never overload the autoclave chamber. Instruments can be placed in sterilization trays or wrapped in paper or muslin before autoclaving.

Arrange the instruments, trays or packs in the autoclave without stacking them. The steam must circulate freely inside the autoclave. Process the instruments as follows:

- Unwrapped instruments at 121°C for 20 minutes at 15 PSI above atmospheric pressure or at 134°C for 3–4 minutes at 30 PSI above atmospheric pressure
- Wrapped instruments at 121°C for 30 minutes at 15 PSI above atmospheric pressure or at 134°C for 15 minutes at 30 PSI above atmospheric pressure

When the autoclaving cycle is complete and the pressure reaches zero, open the door a centimeter or two to allow the steam to escape. Run the drying cycle as recommended by the autoclave manufacturer until all the instruments are dry. Using sterile tongs, remove the instruments. Allow them to cool to room temperature before storing.

Unwrapped items must be used immediately or may be stored in covered, dry, sterile trays for up to a week. Store wrapped packages in a dry, closed cabinet. Instruments remain sterile as long as the wrap is dry and intact.

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